

# NATURE SPRAY PRODUCT MANUAL

\*Please capture wounded area properly every 5 days to see the progress.

Please refer physician or wound team for details wound care management.

## **SIMPLE/BASIC MANUAL (ON PACKAGING)**

1. Cleanse: Gently clean the wound using wet gauze soaked in saline solution/sterile solution and repeat until last piece of gauze seems clean.
2. Apply: Spray the product directly onto the wound adequate to cover all surface area and wait for 5-10 minutes to allow absorption.
3. Cover: Cover wound with 3-4 layers of clean/sterile gauze/pad suitable with wound size and use tape to secure the dressing in place.
4. Repeat step 2 and 3 for at least 3 times daily.

## **DETAILS MANUAL ( FOR PATIENTS TO REFER BASED ON WOUND CONDITION)**

### **WOUND WITHOUT DISCHARGED (TYPE A)**

1. Put on a pair of non-sterile gloves.
2. Gently rub and rinse the wound using wet gauze soaked in saline solution/sterile solution and repeat until last piece of gauze seems clean.
3. Gently pat the wound dry with another clean gauze.
4. Spray the product directly onto the wound enough to cover all surface area of the wound
5. Wait for 5-10 minutes to allow absorption. (Repeat step 6&7 at least 3 times daily)
6. Cover the wound using 3-4 layers of suitable size clean gauze/pad.
7. Use tap to secure the dressing in place. (Do not use bandage)

### **WOUND WITH DISCHARGED (TYPE B)**

1. Put on a pair of non-sterile gloves.
2. Gently rub and rinse the wound using wet gauze soaked in saline solution/sterile solution and repeat until last piece of gauze seems clean.
3. Apply adequate amount of Betadine solution enough to cover the wound.
4. Wipe off excess betadine solution with a piece of clean gauze.
5. Gently pat the wound dry with another clean gauze.
6. Spray the product directly onto the wound enough to cover all surface area of the wound
7. Wait for 5-10 minutes to allow absorption. (Repeat step 6&7 at least 3 times daily)
8. Cover the wound using 3-4 layers of suitable size clean gauze/pad.
9. Use tap to secure the dressing in place. (Do not use bandage)

### **WOUND WITH CAVITY/HOLE (Bedsore or Diabetic foot ulcer)**

1. Follow all steps in Type A or Type B without placing any gauze inside the cavity like wound packing.